

尺寸：250 × 270 mm

颜色： PANTONE 7459U PANTONE 657U White K CMK C 透明

Model: Flat type

Indication for use: For clinical medical personnel to wear,covering the user's mouth,nose and chin. Provides a physical barrier to prevent pathogenic microorganisms,body fluids, particulate matter, etc. from passing directly.

Select the appropriate mask, open the package, and take out the mask to cover the lower half of the face. Stretch the elastic straps behind both ears and adjust the metal nose piece to a better fitting shape by pressing down both sides. For removal of the mask, simply pull off one elastic strap from ear and avoid touching the outer surface of the mask (the contaminated surface).



BIOFIL™

GB 15979-2002

Non-Sterile
Disposable Face Mask
Ear-Loop Style



See manufacturer's instructions.
Size : 17.5×9.5cm
Material: Nonwoven material



Warning

1. Please check the packaging before use. Make sure it is not damaged or broken.
2. The product performance will be degraded after washing. If there is a liquid splash towards the mask, or it make you hard to breathe, please replace it in time.
3. The product cannot be used in an anoxic place where the oxygen concentration is lower than 19.5%, and the harmful substance concentration might immediately threaten life and health.
4. The product is used by one specific person. Repeated use is not suggested.
5. The product shall be disposed according to medical waste regulations and shall not be discarded randomly.

Storage Conditions

Store the product in a clean room with a relative humidity of less than 80% at room temperature, no corrosive gases, cool, dry, and well ventilated.
Date of issue or the latest revision : 2020-03

 **Guangzhou Biofil Air Purification Materials Co., Ltd.**
Room 201, 2ND floor, workshop B, No.1 DouTang Road,
YongHe Development Zone Guangzhou China

Production date:
Exp. date:



250mm